

Message Banking

Message banking allows you to record specific messages in your own voice that can then be played back from within the app. People often choose to use message banking to record names of loved ones, special messages to people and common phrases.

Recording a message bank phrase

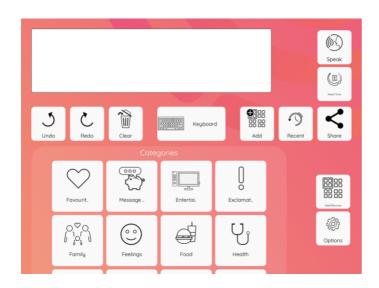

To record a new message, go to the "Message Bank" category.

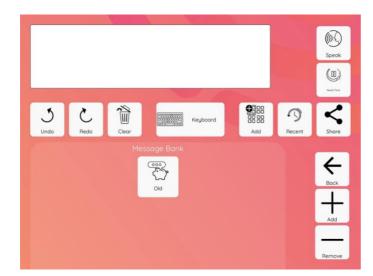


There will be a default category called "Old".


To add new phrases to this category click "**Old**".

To first add a new category, please skip to <u>this section</u>.

Click "Add" to add a new message.



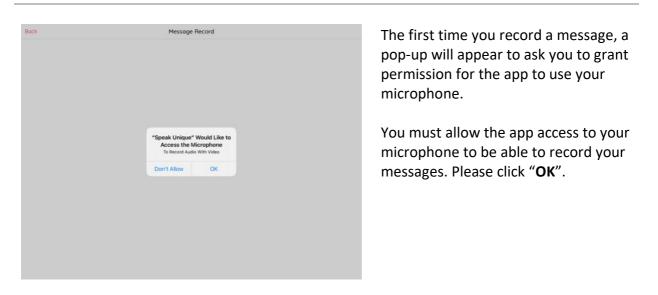

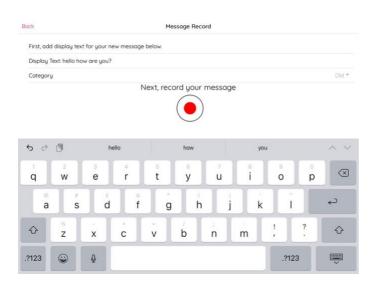

Click in the "**Display Text**" box to bring up the keyboard.

Add a name for the recording. This could be a direct transcript of the message or a title that you will associate with the recording.

In this example, we will store a message that says "Hello, how are you?".

ack	Message Record	
First, add display text fo	r your new message below.	
Display Text: hello how a	re you?	
Category		Old *
	Next, record your message	
	Category	
	Old	
	Cancel OK	
	\sim	

Click the "**Category**" box to select a category for your message bank to go into.

A list of categories will appear. If you have not created any new categories, the only option will be the default category "Old".

Back	Message Record	
First, add display text fo	or your new message below.	
Display Text: hello how	are you?	
Category		Old *
	Next, record your message	
	Category	
	Cid 🖌	
	Cancel OK	
	\smile	

Message Record

Next, record your message

Select the category you would like to add the phrase to. The name of the selected category will turn pink and a tick will appear on the right hand side. Then click "**OK**".

In this example the "**Old**" category has been selected.

When you are ready to record your message, press the red circle and speak your message out loud.

The red circle will change to a red square and a timer will start to indicate that the recording is in progress.

Back

Category

First, add display text for your new message below.

Display Text: hello how are you?

ock	Message Record	
First, add display text for your new m	essage below.	
Display Text: hello how are you?		
Category		Old *
	00:02.596	
	Next, record your message	
Save	Previev	v

When you have finished recording your message press the red square to stop the recording.

A **"Save**" and **"Preview**" button will appear when you stop recording.

If you would like to listen back to the recording please press "**Preview**".

First, add display text f	or your new message below.	
Display Text: hello how	are you?	
Category		Old *
	00:02.596	
	Next, record your message	
	Would you like to rerecord your message?	
	Yes No	

If you would like to rerecord your message, click on the red circle again.


A message will appear asking you to confirm that you would like to rerecord your message.

If you choose to rerecord the message, the original file will be replaced.

When you are happy with the recording, click "**Save**".

Back	Me	ssage Record	
First, add display text	for your new message below.		
Display Text: hello how	are you?		
Category			Old *
	0	0:02.596	
	Next, reco	ord your message	
	Save	Preview	

	Recording Saved display text for your new message below. xt: click here to enter your display text		A confirmation message will appear to show that the recording has been
Category	Next, record your message	Old ~	saved.
	Recording Saved display text for your new message below. xt: click here to enter your display text Next, record your message	Old *	You can continue to add more messages. When you are finished recording, press the " Back " button in the top left corner to return to the category page.

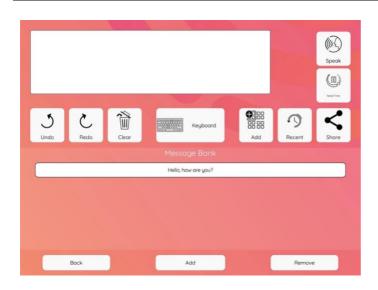

To remove a message bank phrase, click "**Remove**".

Select the phrase(s) that you would like to remove. Selected phrases will be highlighted in grey.






Click "**Remove**" to remove the phrase.





A message will appear asking you to confirm if you would like to remove the phrase. Click "**Yes**" to confirm you would like to delete the message bank phrase.

The phrase will be removed from the list. Click "**Back**" to return to message bank category.

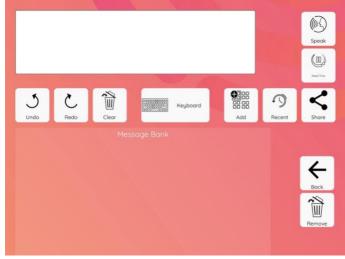


Add a new message bank category

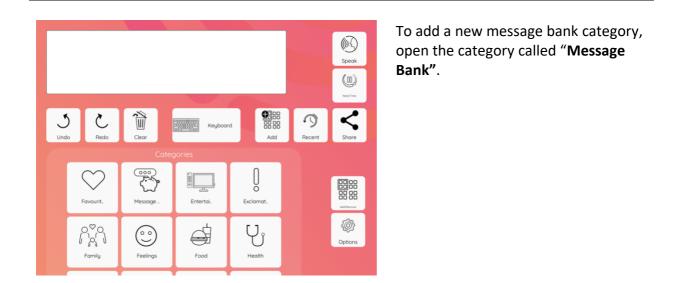

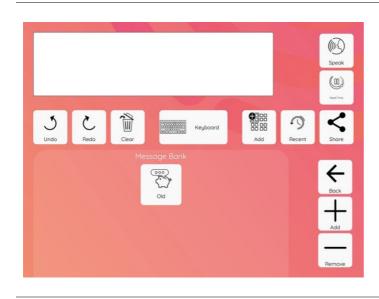

Click "Add" on the right hand side.


Click in the box "Name of new
 category" to pull up the keyboard.

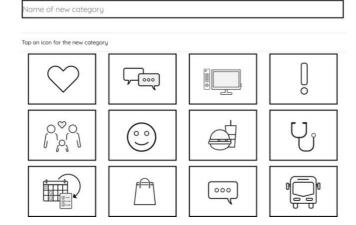

Add to Message Bank

Back

Please enter a category name

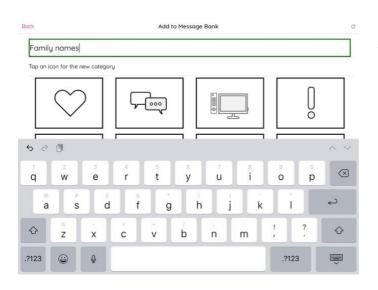

Add to Message Bank

• 2022 ¢

000

5-600

 (\circ)

-A

Enter the name of the category you would like to create.

In this example, a category called "Family Names" will be created.


Select an icon for your category. The selected icon will have a green box around it. Then click "**Add**".

0

0

Ųj

In this example, the icon of a love heart has been selected.

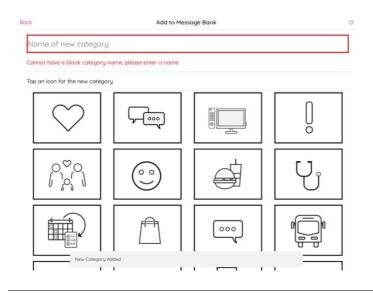

A confirmation message will then be displayed to show the new category has been added.

Back

Family names

Tap an icon for the new category

000





You can continue to add new categories or click "**Back**" to return to message bank category.


Your new category will now show within the "Message Bank" category on the app.

To add phrases to a new category, please follow the <u>steps above</u>.



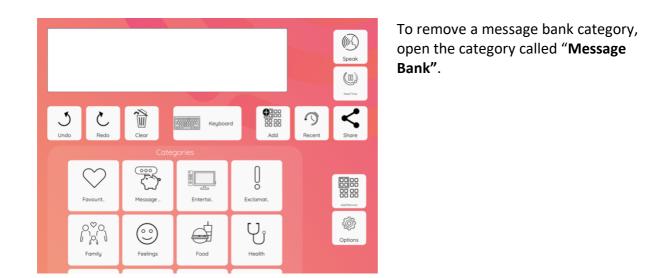

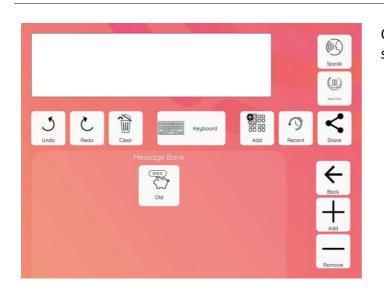

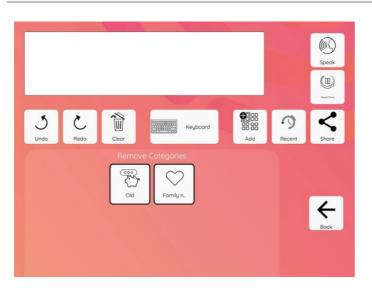


Click "**Remove**" on the right hand side.

Select which category you would like to remove.



move selected Message Categories? The selected category will be outlined in green.

Click "**Remove**" on the right hand side.

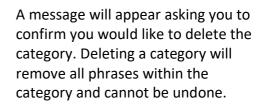


(())

Speal

5

If you would definitely like to delete the category and all the phrases press "Yes".

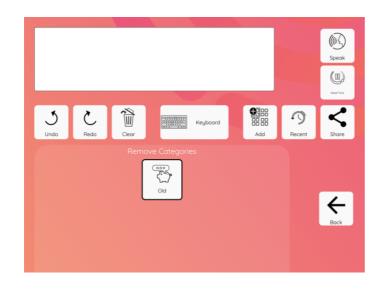

The category will be removed. Click "**Back**" to return to the message bank category.

J

Sync message bank phrases across your devices

You can sync your message bank phrases across multiple devices - for example, if you have recorded all your message bank messages on your tablet you can sync them, so they also appear on your phone.

In this example, we will show messages that were recorded on a tablet (the first device), being copied over to a phone (the second device).

Speak 3 C Ì J \mathbb{C} 5-000 Chat Message Entert 6 Ő đ \bigcirc Feeling Food Exclam Family

Uploading messages from your first device

Go to "Options".

In this example the first device is a tablet.

Bock	Options	,
Font		>
Reset Accessibility Settings		
Categories and Phrases		
Sync Account Data		>
Rearrange Categories		>
Rearrange Phrases		>
Edit Take Your Time Phrases		>
Smart Assistant Category		>
Manage Message Bank		
Sync Message Bank		>
Categorise Message Bank		>
Speech Settings		
Voice Speed and Volume		>
Pronunciation Exceptions		>
Reset Speech Settings		

Select "Sync Message Bank".

Back Sync Message Bank If you are using the SpeakUnique app on more than one device, you can sync your banked messages across the app. If you want to upload messages on this device for use on a different device - click upload Upload If you want to download messages from a different device for use on this device - click download. Download	On the first device, click " Upload ". In this example we are uploading messages from a tablet.
Back Upload Message Bank To add messages from this device onto a different device, you need to first upload the recordings. Then, you will be able to download the messages onto your second device. Please select the messages you would like to upload from the list below, or upload all the messages. Upload all Upload selected CURRENT MESSAGE BANKED Helol How are you?	
Back Upload Message Bank To add messages from this device onto a different device, you need to first upload the recordings. Then, you will be able to download the messages onto your second device. Please select the messages you would like to upload from the list below, or upload all the messages. Upload all Upload selected CURRENT MESSAGE BANKED Helo! How are you?	If you choose to select specific messages, click on each message to select them. Selected messages will show with a tick on the right side. In this example both "Hello!" and "How are you?" are selected.
Back Upload Message Bank To add messages from this device onto a different device, you need to first upload the recordings. Then, you will be able to download the messages onto your second device. Please select the messages you would like to upload from the list below, or upload all the messages. Upload all Upload selected	Press " Upload Selected ".

0

Hello! How are you?

ck		Upload Message Bar	ık
			device, you need to first upload the messages onto your second device.
Please select the the messages.	messages you	u would like to uploa	ad from the list below, or upload all
U	pload all		Upload selected
CURRENT MESSAGE BANKED		_	
Hello!		Colording	
How are you?			

A spinning circle timer will appear while the messages are being uploaded.


A confirmation message will appear to show that the messages have been uploaded. Press "**Okay**".

To add messages from this device onto a different device, you need to first upload the recordings. Then, you will be able to download the messages onto your second device. Please select the messages you would like to upload from the list below, or upload all the messages. Upload all Upload selected CURRENT MESSAGE BANKED Your messages are uploaded. You can now download these on your other devices Held Your messages ore uploaded. You can now download these on your other devices	Back	Upload Message Bank	
recordings. Then, you will be able to download the messages onto your second device. Please select the messages you would like to upload from the list below, or upload all the messages. Upload all Upload selected Your messages are uploaded. You can now download these on your other devices	DUCK	Oploda Message Bank	
the messages. Upload all Upload selected current message bankeb Your messages are uploaded. You can now download these on your other devices Image: Selected			
CURRENT MESSAGE BANKED Your messages are uploaded. You can now download these on your other devices		would like to upload fro	m the list below, or upload all
CURRENT MESSAGE BANKED uploaded. You can now download these on your other devices	Upload all	Up	bload selected
How are now?		uploaded. You can now download these on your	
How are you?		other devices	V
		Okay	

Downloading messages onto your second device

Once you have uploaded the messages from the first device, you can then download them onto your second device. In this example, we have uploaded the messages from a tablet, and we are now going to download the messages onto a phone.

Complete the following steps on your second device that you would like to download the messages onto:

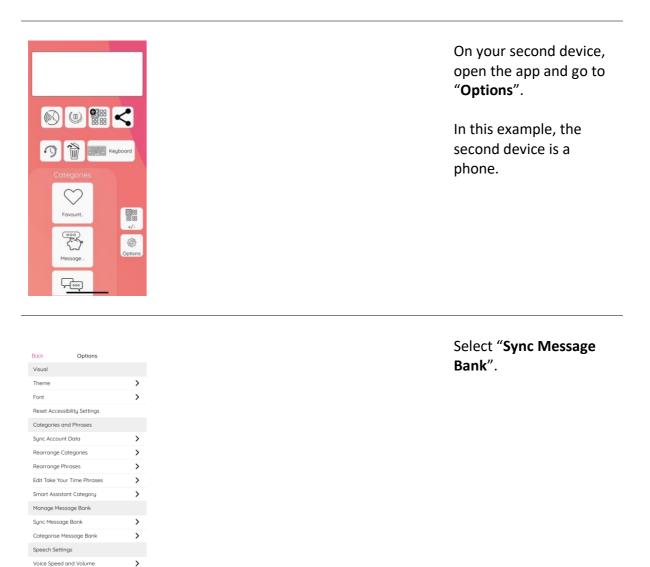

Pronunciation Exceptions

Danat Canach

>

Select "Download"

If you are using the SpeakUnique app on more than one device, you can sync your banked messages across the app.

Sync Message Bank

Back

If you want to upload messages on this device for use on a different device - click upload

If you want to download messages from a different device for use on this device - click download.



Back Download Message B...

How are you?	

Bock Download Message B...

Once you have uploaded messages from one device, you can then download them onto this device. Once the messages have been downloaded you will be able to use them on this device.

Please select the messages you would like to download from the list below, or download all the messages.

CURRENT MESSAGE BANKED

Hello!

How are you?

Dow

You can either use the "Download All" button to download all of the messages you uploaded, or you can download specific messages using the "Download Selected" button.

In this example, we will use the "**Download Selected**" button.

Select the messages you wish to download. Selected messages will show with a pink tick next to them. Once you have selected all the messages you would like to download, click "Download selected".



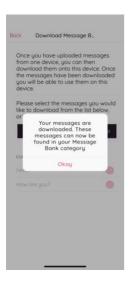


Image: Sector Secto

A "Downloading" window will show as the banked messages are being downloaded.

You will receive a confirmation message once the banked messages have been downloaded successfully. Press "**Okay**".

The messages will then show in the "Message Bank" category on your second device.