

Instructions for Creating a Voice Build

This guide gives you set by step instructions for creating a Voice Build with SpeakUnique.

Table of Contents

<i>Instructions for Creating a Voice Build</i>	1
<i>Before you begin</i>	2
<i>Device compatibility</i>	2
SpeakUnique App	2
Speed Test App	2
Third Party Apps for iOS	2
Windows Devices	3
<i>Registering for an account</i>	4
<i>Completing the Recording</i>	8
Recording Set-Up	8
Microphone Permissions	11
The Recording Screen	12
The Screen During Recording	13
<i>Listening to Your Voice Build</i>	16
<i>Payment and Funding for Voice Build</i>	18
<i>Downloading Your Voice Build</i>	20

Before you begin

To record your voice with SpeakUnique, you will need:

1. A laptop or computer. Recording is not currently supported on iPads and tablets.
2. An external headset microphone that connects to your computer via the USB port or headphone socket. We do not recommend a specific headset, but headsets made by Sennheiser, Plantronics and Logitech are reliable.
3. A quiet room with no background noise and ideally soft furnishings to remove any echoing.

Device compatibility

Before you start recording, please ensure the device you want to use your voice on (e.g. communication aid, phone or tablet) is compatible with SpeakUnique voices. SpeakUnique voices can be downloaded unlimited times across a range of devices.

SpeakUnique App

The SpeakUnique app is a text-to-speech app that allows you to input text via a keyboard or pre-set categories and phrases. This typed message is then generated in your personalised synthetic voice. The app can be used with various input methods, such as switch access. It is compatible with iOS devices (i.e. iPhones and iPads) running **iOS 12** or higher, and many Android devices, depending on the chip used by the particular device.

A current list of tested Android devices is available here:

<https://www.speakunique.co.uk/session/using-your-voice-speakunique-app-android>.

You can check whether a given device is compatible or not on the list.

Speed Test App

If you already own an Android device but you're not sure if it's compatible, you can check this by using our speed test app, which can be downloaded free from the Google Play store by searching for "SpeakUnique Speed Test". The app will run a short speed test, which takes less than a minute, to confirm if your device is compatible with the SpeakUnique app. Once you have run the test, you can delete the speed test app from your device.

Third Party Apps for iOS

We have partnered with Smartbox and Therapy Box to integrate SpeakUnique voices into Grid for iPad and Predictable for iOS devices, respectively. A licence for Grid for iPad or Predictable must be purchased separately or provided through your healthcare provision. Please note SpeakUnique voices **do not** integrate into Predictable on Android devices.

Windows Devices

For Windows devices, the voices are provided in SAPI5 format where they can be accessed by any software on the device which relies on synthetic voices.

We do not offer our own SpeakUnique app for Windows devices – you will need to use third party software to access your voice.

We have confirmed the voices are compatible with the speech generating devices listed below but please get in touch if we have not included a device you intend to use the voice on.

Tobii Dynavox

Indi 7, i-12+, i-15+, i-13, i-16, I-110

SmartBox


Grid Pad 8, Grid Pad 10, Grid Pad 11, Grid Pad 12, Grid Pad 13, Grid Pad 15, Touch Pad

Liberator/ PRC


Accent 1400, Accent 1000 and Accent 800

Registering for an account

Visit www.speakunique.co.uk and click **Sign Up/ Login**.

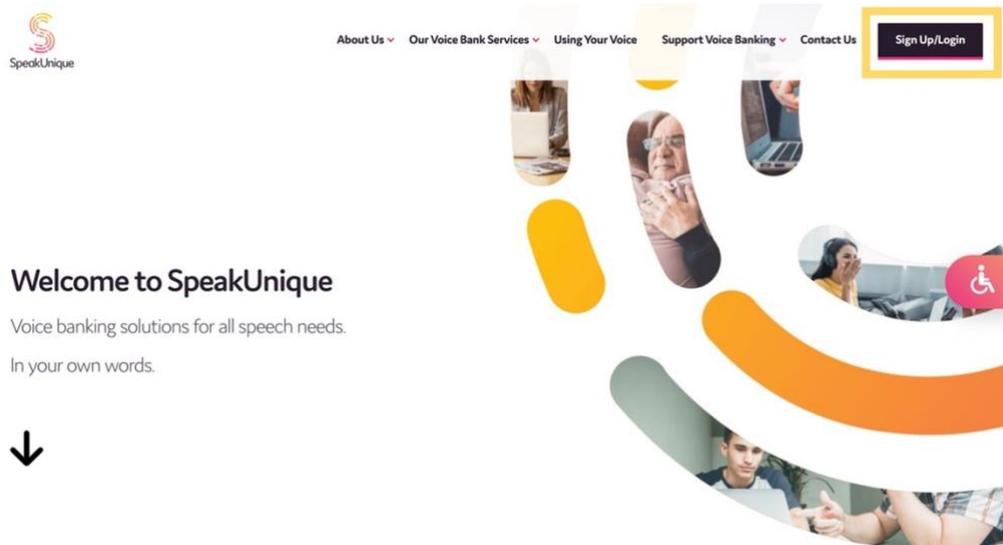

To register for a new account, please enter your **email address** and click **Begin Registration**

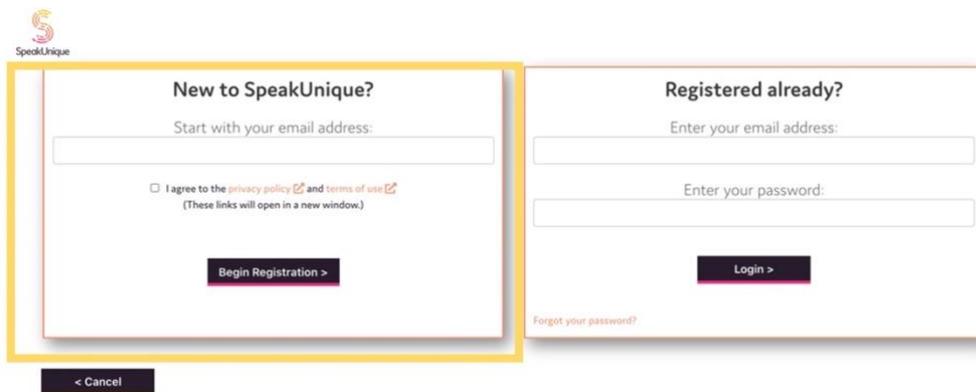

Enter your first name, last name and choose a password. We'd love to know how you found out about us and you can decide if you would like to be kept up to date on our news through our mailing list! Then click **Continue**.



Let's get to know each other:

Fields marked with a red star * are required for registration.

First name *

Last name *

Choose a password *

How did you hear about SpeakUnique?

Would you like to join our marketing list?

[← Cancel](#) [Continue →](#)

This guide will show you how to create a **Voice Build**, click **Get Started**.



Please select one of our voice bank services:

<p>Voice Build</p>  <p>£175</p> <p>If your speech is not yet affected. A synthetic replica of your own voice.</p> <p>Get Started</p>	<p>Voice Repair</p>  <p>£250</p> <p>If your speech is already affected. For if your voice is slow, slurred, or the pitch has changed. A repaired version of your voice.</p> <p>Get Started</p>	<p>Voice Design</p>  <p>£300</p> <p>Use old recordings of your voice. OR Create a bespoke voice from voice donors.</p> <p>Get Started</p>	<p>Healthcare Professionals</p>  <p>Free trial of Voice Build for healthcare professionals. Please register with your professional email address.</p> <p>Get Started</p>
------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------	------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------

Not sure? [Find out more about our services here](#) or contact our support team at support@speakunique.co.uk

Please **complete your personal details**. We ask for this information to help us better understand how to build your synthetic voice. You do not need to provide us with your medical condition if you do not want to.



Let's get to know each other

We collect this information to help us create your synthetic voice.

What is your medical diagnosis or condition?

When were you diagnosed?
 Month... Year...

Please select your date of birth*
 Day... Month... Year...

Please select your gender*
 Male Female Other

← Previous

Please note that you do not need to provide the information set out on this page and you can proceed to the next page without completing these fields. However, in order to improve our understanding of how to null your voice, you can complete the information on this page.

Next →

For the purposes of applicable data protection law, SpeakUnique Limited ("SpeakUnique", "we", "us") is a controller of personal data you provide by submitting this form. By completing this form, you consent to your personal data being collected and processed by us in order to improve our understanding of how to build, repair or design your voice in accordance with our Privacy Notice at www.speakunique.co.uk/privacy-policy. You can withdraw your consent at any time by emailing privacy@speakunique.co.uk.

If you are being supported to use SpeakUnique by a **Healthcare Professional**, you can add their details to your account to grant permission for them to liaise with us on your behalf and allow them to view your progress. If you are unsure of this information at this stage, you can add this at any time from within "My Account".

Your Healthcare Professional

Healthcare Professional Name:

Healthcare Professional Email Address:

Healthcare Professional Phone Number:

Job Role:

Your Healthcare Professional will be able to:

- View your name and email address
- View your recording progress
- Listen to the recordings you complete with SpeakUnique or listen to the recordings you upload to your account
- Listen to samples of your personalised synthetic voice
- Discuss the status and any feedback on the voice with the SpeakUnique team
- Apply for funding on your behalf

By ticking this box you confirm you give permission for SpeakUnique to liaise with your named healthcare professional about your account. You can withdraw your consent at any time by emailing support@speakunique.co.uk.

← Cancel

Save Changes →

Please let us know how you would describe your accent. The **Accent Influence** box can be used if there is any extra information about your accent that you think would be helpful to tell us.

How would you describe your accent?*

* this information is required

We use information about your accent to better understand how to generate your synthetic voice and make sure the voice sounds as much like you as possible, so it's important this information is correct.

If English is not your native language, you might feel it is more accurate to select the country of your native language to describe your accent. You can always add more information about accent influences in the free text box below.

Country:

Please select

Region/County/State:

Please select

Do you have any other accent influences? [More about this ?](#)

(optional)

Your registration is now complete, and you can continue to the recording now. You will receive an email to confirm your account has been created.



Thanks for signing up to SpeakUnique.

Your account has successfully been created.

A confirmation email has been sent to the email address you provided. Please use this email address and the password you set at registration to log back in to your account.

You are now ready to start creating your synthetic voice build.

[Start Now →](#)

Completing the Recording

Recording Set-Up

The next screens contain tips on how to complete a good recording.

We have prepared a short video explaining the recording process.



Recording your voice

For step-by-step instructions on how to record your voice, please watch our video below



< Previous

Next >

Recording tips are also provided in written format.



Recording Tips

Our state-of-the-art speech synthesis technology works by mimicking what it hears in your recording.
This is what makes it so great at capturing your identity in the voice.

But there are a few things to remember:

Make sure you are in a quiet room with no background noise. For example, if you have any pets make sure they're out of the room, switch off any appliances like the dishwasher or washing machine, and try to record away from any busy streets.

The quality of the audio will also affect the quality of the voice. This is why we ask you to **use an external microphone** (not the one built-in to your computer). Try not to record in a room that is very echoey: rooms with lots of soft furnishings are good, and closing windows and curtains can also help.


Read the sentences in a natural voice - don't over-emphasise any of the words but try not to sound too flat. Otherwise this will come through in your synthetic voice.

Only record when you are reading the correct sentence. Try not to cough or stutter on a word during the recording. If you do or **if you make a mistake while reading a sentence then re-record the sentence.**


< Previous

Next >

Make sure your external microphone is connected to your computer and is positioned in line with your jaw. You do not want the microphone too close to your mouth as this can lead to a poor-quality recording.

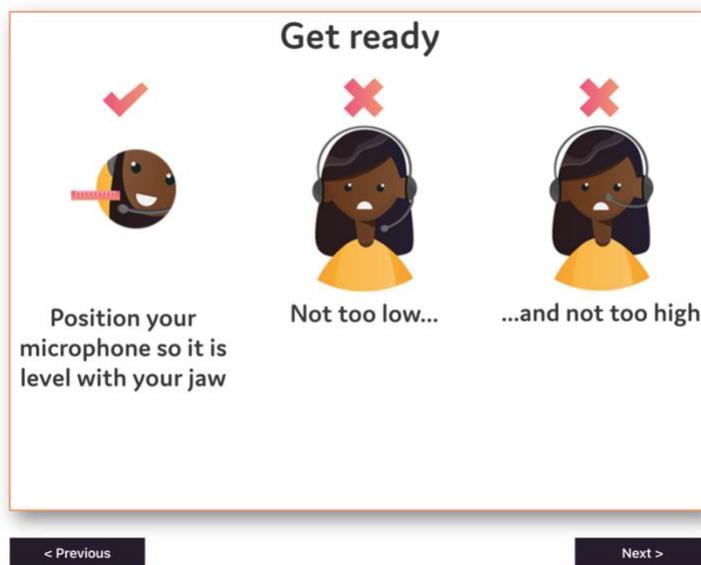

We offer a selection of different scripts for you to choose from. First select what genre of book you would like to read from; each genre contains a range of titles to pick from.

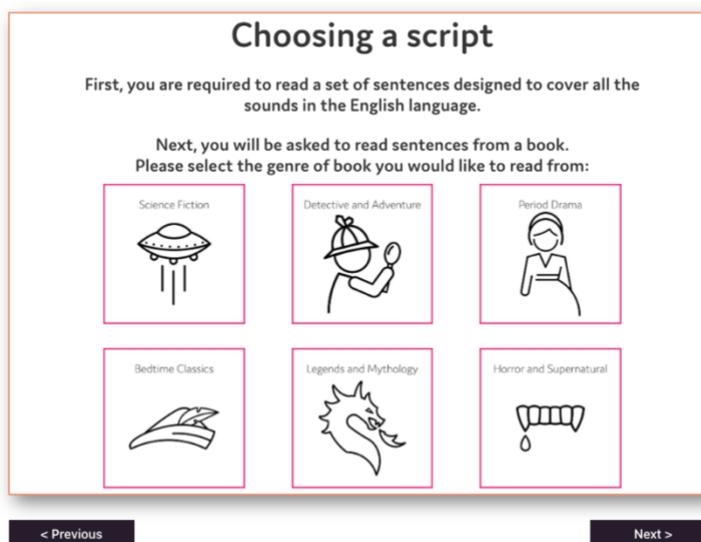

Once you have selected a genre, you will see the available titles. Each title has been rated for ease of reading. Use the circle button on the right to select which title you would like to read.

We also have a “Simple Script” available which is made up of shorter sentences and idioms which we can make available to you, if you struggle to read any of the provided scripts. Please contact us at support@speakunique.co.uk if you would like to use this script.



Choosing a script

You have selected **Bedtime Classics**. Please select a book to read from below.

Title	Author	Reading Difficulty	
Alice in Wonderland	Lewis Carroll	Average	<input type="radio"/>
Dick Whittington and His Cat	Edith Carrington	Easier	<input type="radio"/>
Peter Pan	J.M. Barrie	Harder	<input type="radio"/>
Stories by Beatrix Potter	Beatrix Potter	Average	<input type="radio"/>
The Jungle Book	Rudyard Kipling	Average	<input type="radio"/>

Please note that some books may have been edited for clarity and ease of reading. The books have been rated for difficulty based on factors such as length of sentences, structure of sentences and vocabulary. To return to the genre selection please press Previous

< Previous Next >

You are now ready to start recording. Please make sure you are comfortable.



Get Comfortable

Set yourself up somewhere quiet and comfortable where you can keep your computer stable. Grab yourself a glass of water or a cup of tea.



< Previous Next >


Microphone Permissions

Before you begin recording, you need to ensure the correct microphone is selected. A list of available microphones on your device will be shown on screen, please choose your microphone and click **Continue**.

We need to access your microphone to continue. If there is a box above asking for permission to use your microphone, please click "Allow".

If you have done that and you are still seeing this message, please [reload this page](#).

If there is no box and you are having trouble, look for a small microphone icon in the URL bar at the top of the window and try clicking it to reset your permissions for this page.

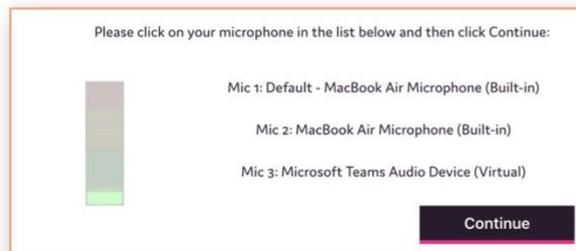

If you complete the steps outlined in the screenshot above and the microphone is still not recording, please check the microphone permissions on your web browser and your laptop/computer and ensure you have granted access to www.speakunique.co.uk to access your microphone.

The Recording Screen

The screenshot shows the SpeakUnique recording interface. At the top left is the SpeakUnique logo. At the top right, there is a 'Your Account' section with a 'Logout' button. Below the logo, the status is 'Ready to Record' and the microphone is identified as 'MacBook Air Microphone (Built-in)' with a 'Change' link. The main recording area contains a sentence 'Please call Stella.' with a volume bar on the right. Below the sentence are navigation buttons: a left arrow, a 'Play' button, a 'Record' button, and a right arrow. A progress bar at the bottom shows '1%' completion and 'This is sentence 1 of 125 (minimum recordings)'. At the bottom left, there is a 'Save and Log Out' button and a note about saving progress. At the bottom right, there is a 'Reading: Dracula' section with a '[Click to change book]' link.

1 Please call Stella.

2 Record

3 Play

4

5

6

8 Save and Log Out

1 %

Reading: Dracula
[Click to change book]

Status: Ready to Record

Microphone: MacBook Air Microphone (Built-in) Change

Your Account Logout

This is sentence 1 of 125 (minimum recordings).

If you want to complete the recording at a later date please click this button. Your current progress will be saved and you will be logged out.

1. This is the sentence you need to read aloud.
2. Press **Record** to start recording each sentence.
3. The **Play** button allows you to listen to the sentence you have just recorded - please listen back to sentences frequently to check the quality.
4. The **volume bar** indicates the volume of the recording. An error message will appear if the recording is too loud, too quiet or if your microphone is too close to your mouth.
5. The **arrows** allow you to move between sentences. You can also use the arrow keys on your keyboard to do this.
6. The **progress bar** shows the sentence number you are reading and what percentage of the recording you have completed.
7. Here you can change the external **microphone** you want to use. Please note the name of the microphone might differ from what you expect.
8. Use the **Save and Log Out** feature if you would like to save your recording.

The Screen During Recording

SpeakUnique

Your Account Logout

Status: Ready to Record Microphone: External Microphone (Built-in) Change

1 'You had better ask him,' said the father, giving his little girl a kiss before he went out of the room. 3

2

Play Record

This is sentence 127 of 150 (total recordings).

85 %

Reading: Dick Whittington and His Cat [Click to change book]

Save and Log Out If you want to complete the recording at a later date please click this button. Your current progress will be saved and you will be logged out.

Save and Submit If you have recorded all the sentences you can manage please click to submit your recording and we will generate your synthetic voice.

4

1. Press record and start speaking. If you make a mistake, you are interrupted, you notice background noise or if you cough during the recording please rerecord the sentence.
2. Press **Stop** to stop recording the sentence.
3. A red box will appear around the screen to indicate that recording is in progress.
4. Your synthetic voice can be built on a minimum of 125 sentences and a maximum of 150 sentences.

The **Save and Submit** button will appear once you have reached sentence 126 and can be used when you are ready to submit your recordings to create your synthetic voice. If you wish to record all 150 sentences, please continue recording until you reach the end of the process.

After you've clicked **Save and Submit**, please confirm your **account type** and **accent information**. The submission will not be complete and the synthetic voice will not be generated until you click **Submit**.

Please ensure you have registered for the correct service – if you wish to change from a Voice Build to a Voice Repair (or vice versa) you can do this now.

Once you have submitted your recordings, your account will show that your voice is being generated.

The screenshot displays a user account interface for Alice Smith. At the top left is the SpeakUnique logo. The user's name, Alice Smith, is prominently displayed. In the top right corner, there are links for 'Your Account' and a 'Logout' button. Below the name, the email address <alice@speakunique.co.uk> is shown. The main content area is divided into two panels: 'Your SpeakUnique Products' and 'Your Details & Preferences'. The 'Your SpeakUnique Products' panel shows a 'Voice Build' product with a status message: 'Your recordings have been submitted. Thank you! We will send you an email when your synthetic voice is ready to listen to.' The 'Your Details & Preferences' panel lists personal information: Name: Alice Smith, Email: alice@speakunique.co.uk, Date of Birth: 17/06/1994, and Marketing Information: No. An 'Update' button is located at the bottom of this panel. At the very bottom of the page, there is a footer with the text: 'Need additional help? Contact our support team at <support@speakunique.co.uk>'.

SpeakUnique

Alice Smith

Your Account **Logout**

<alice@speakunique.co.uk>

Your SpeakUnique Products

Product: Voice Build

Status: Your recordings have been submitted. Thank you!
We will send you an email when your synthetic voice is ready to listen to.

Your Details & Preferences

Name: Alice Smith
Email: alice@speakunique.co.uk
Date of Birth: 17/06/1994
Marketing Information: No

Update

Need additional help? Contact our support team at <support@speakunique.co.uk>

Listening to Your Voice Build

You will be notified by email once your synthetic voice has been generated. If you do not receive an email, please check your junk folder or log back into your account to check your voice status.

Log into your account at www.speakunique.co.uk and click **Download/listen to samples of your synthetic voice.**

The screenshot shows the user account page for Alice Smith. At the top left is the SpeakUnique logo. The user's name, Alice Smith, is displayed prominently. To the right, there are links for 'Your Account' and a 'Logout' button. Below the name, the email address <alice@speakunique.co.uk> is shown. The page is divided into two main sections: 'Your SpeakUnique Products' and 'Your Details & Preferences'. The 'Your SpeakUnique Products' section shows the product 'Voice Build' with a status of 'Your Voice Build has been generated.' and a button to 'Download/Listen to your synthetic voice'. The 'Your Details & Preferences' section lists personal information: Name: Alice Smith, Email: alice@speakunique.co.uk, Date of Birth: 17/06/1994, and Marketing Information: No. An 'Update' button is located at the bottom of this section.

You can listen to 5 samples of your synthetic voice.

The screenshot shows a confirmation page titled 'Your Voice Build is ready!'. It features five audio samples, each with a text description and a 'Play' button. The samples are: 1. 'Here is a sample of my personalised SpeakUnique voice.' 2. 'This is based on a recording of my voice.' 3. 'It means I can continue to communicate in my own voice.' 4. 'The quick brown fox jumped over the lazy dog.' 5. 'I'll have the soup of the day, please.' Below the samples, a message says 'Isn't it great? We hope you love your voice as much as we do!'. A section titled 'What next?' explains that users need to purchase or apply for funding to download the voice. At the bottom right, there are two buttons: 'Apply For Funding' and 'Pay Now'. A 'Show More' link is located at the bottom left.

If you have any feedback on the voice, you can use our **Thumbs Up/Thumbs Down** system to let us know what you think. If you have more specific feedback or would like to upgrade to our **Voice Repair** service, please email support@speakunique.co.uk.

What next?

If you're happy with your samples and want to start using your voice, you will need to purchase or apply for funding for your SpeakUnique voice in order to be able to download it.

Use the Apply for Funding button to apply for funding from one of our charity partners. If you wish to purchase your voice yourself, please use the "pay now" button. You can also email funding@speakunique.co.uk if you would like to discuss funding from the health service or a charity not listed on our website.

[Hide](#)

Apply For Funding

Pay Now

What do you think?

If you're happy with your voice but don't need to download it yet we will store the voice for as long as you like - free of charge! Click  to let us know you like the voice but don't need it yet!

If you don't like the voice, please don't apply for funding or purchase the voice just yet. Just click  and a member of our team will be in touch with you or email support@speakunique.co.uk.

[Show More](#)



Payment and Funding for Voice Build

You can opt to self-fund the voice via a secure online payment by clicking **Pay Now**. If you wish to complete a funding application, please click **Apply for Funding** and follow the below steps.

Your Voice Build is ready!

"Here is a sample of my personalised SpeakUnique voice." 

"This is based on a recording of my voice." 

"It means I can continue to communicate in my own voice." 

"The quick brown fox jumped over the lazy dog." 

"I'll have the soup of the day, please." 

Isn't it great? We hope you love your voice as much as we do!

What next?

If you're happy with your samples and want to start using your voice, you will need to purchase or apply for funding for your SpeakUnique voice in order to be able to download it.

Apply For Funding
Pay Now

[Show More](#)

Our current funding partnerships will be displayed, alongside eligibility criteria.

A selection of charities will fund a SpeakUnique voice for communication aids. Please see below for the charities that will currently fund our voices and the eligibility criteria for each fund.

 <p>Irish Motor Neurone Disease Association Irish Motor Neurone Disease Association</p> <p>MINDA are able to offer funding for Voice Build, Voice Repair or Voice Design, to those living with a confirmed diagnosis of MND in the Republic of Ireland. To apply for funding from MINDA you will need to be registered with MINDA and provide contact details for your Speech and Language Therapist.</p> <p style="text-align: center;">Apply →</p>	 <p>MND Scotland Making time count Motor Neurone Disease Scotland</p> <p>MND Scotland will fund 100% of the cost of SpeakUnique products for those resident in Scotland who have a confirmed diagnosis of Motor Neurone Disease or Kennedy's Disease.</p> <p style="text-align: center;">Apply →</p>	 <p>mnda motor neurone disease association Motor Neurone Disease Association</p> <p>The MND Association will fund 100% of the cost of SpeakUnique products for those resident in England, Northern Ireland or Wales, who have a confirmed diagnosis of Motor Neurone Disease or Kennedy's Disease.</p> <p style="text-align: center;">Apply →</p>
 <p>Multiple System Atrophy Trust Multiple System Atrophy Trust</p> <p>The MSA Trust can offer funding for their members living with MSA in the UK and Ireland. Please make sure you have checked with the Trust that you are eligible and that you have the support of a Speech and Language Therapist or MSA Health Care Specialist.</p> <p style="text-align: center;">Apply →</p>	 <p>PSPA Progressive Supranuclear Palsy Association</p> <p>PSPA are able to offer funding for SpeakUnique products for those who live in the UK with a confirmed diagnosis of PSP or CBD. Please note you will need to provide contact details of a healthcare professional to apply.</p> <p style="text-align: center;">Apply →</p>	 <p>ATAXIA Ataxia UK</p> <p>Ataxia UK can offer funding for Voice Build, Voice Repair and Voice Design for people with ataxia who live in the UK.</p> <p style="text-align: center;">Apply →</p>
 <p>Mouth Cancer FOUNDATION Awareness and Support Mouth Cancer Foundation</p> <p>The Mouth Cancer Foundation will offer funding for Voice Build, Voice Repair and Voice Design for patients, who live in the UK and are at risk of losing their voice through surgery and treatment of head and neck cancers.</p> <p style="text-align: center;">Apply →</p>	 <p>MND and Me FOUNDATION MND and Me</p> <p>The MND and Me Foundation will offer funding for Voice Build, Voice Repair and Voice Design for patients, who live in Queensland Australia and have a confirmed diagnosis of MND. If not already registered with the Foundation, a Client Contact Form and Consent Form will be required.</p> <p style="text-align: center;">Apply →</p>	

The steps for applying for funding are specific to each charity but the required information will be shown on screen.

For example, the **MND Association** application is shown below.

Apply for MND Association Funding

To apply for funding to cover the cost of your SpeakUnique voice from the MND Association please complete this form:

Name*	<input type="text"/>
Address*	<input type="text"/>
Postcode*	<input type="text"/>
Phone number*	<input type="text"/>
Email address*	<input type="text"/>
Date of Birth*	<input type="text"/>
Speech & Language Therapist (SLT) details (if applicable)	<input type="text"/>
Doctor/GP details*	<input type="text"/>
Date of Diagnosis*	<input type="text"/>

Applications from some charities require endorsement from a healthcare professional. You are required to provide the email address of the healthcare professional endorsing your application for funding. An automatic email will be sent to them explaining what they need to do.

For example, the **PSPA** application is shown below.

Apply for PSPA Funding


To apply for funding to cover the cost of your SpeakUnique voice from PSPA please complete this form.

The details you enter will be sent to the named healthcare professional and PSPA, alongside the message below asking the healthcare professional to endorse your application for funding from PSPA. We will notify you by email on the outcome of your funding application.

<table style="width: 100%;"><tr><td>Name*</td><td><input type="text"/></td></tr><tr><td>Phone number*</td><td><input type="text"/></td></tr><tr><td>Email address*</td><td><input type="text" value="alice@speakunique.co.uk"/></td></tr><tr><td>SpeakUnique Product</td><td><input type="text" value="Voice Build"/></td></tr><tr><td>Cost</td><td><input type="text" value="£175"/></td></tr><tr><td>Name of healthcare professional endorsing application for Support Grant*</td><td><input type="text"/></td></tr><tr><td>Contact details for healthcare professional*</td><td><input type="text"/></td></tr></table>	Name*	<input type="text"/>	Phone number*	<input type="text"/>	Email address*	<input type="text" value="alice@speakunique.co.uk"/>	SpeakUnique Product	<input type="text" value="Voice Build"/>	Cost	<input type="text" value="£175"/>	Name of healthcare professional endorsing application for Support Grant*	<input type="text"/>	Contact details for healthcare professional*	<input type="text"/>	<p>Below is a preview of the email that we will send on your behalf to your healthcare professional</p> <div style="border: 1px solid #ccc; padding: 5px;"><p>Date: Thu, 18 Mar 2021 To: From: SpeakUnique <support@speakunique.co.uk> Subject: PSPA funding request for a SpeakUnique voice</p><p>[Your name] has applied for funding through PSPA for the cost of a SpeakUnique voice. They have nominated you as a healthcare professional to support this application. Please respond to this email, ensuring PSPA is cc'ed, confirming that you endorse this application and give a short overview of why this service would be beneficial to the individual.</p><p>If the individual applying for funding is a resident in Scotland, please give a reason why the SpeakUnique voice cannot be funded through NHS Scotland's provision of communication aids.</p><p>Details of the individual applying for funding: Name: Phone number: Email address: alice@speakunique.co.uk SpeakUnique product: Voice Build Cost: £175</p></div>
Name*	<input type="text"/>														
Phone number*	<input type="text"/>														
Email address*	<input type="text" value="alice@speakunique.co.uk"/>														
SpeakUnique Product	<input type="text" value="Voice Build"/>														
Cost	<input type="text" value="£175"/>														
Name of healthcare professional endorsing application for Support Grant*	<input type="text"/>														
Contact details for healthcare professional*	<input type="text"/>														

Downloading Your Voice Build

The **Download Now** button will appear once your voice has been purchased or your funding has been approved.

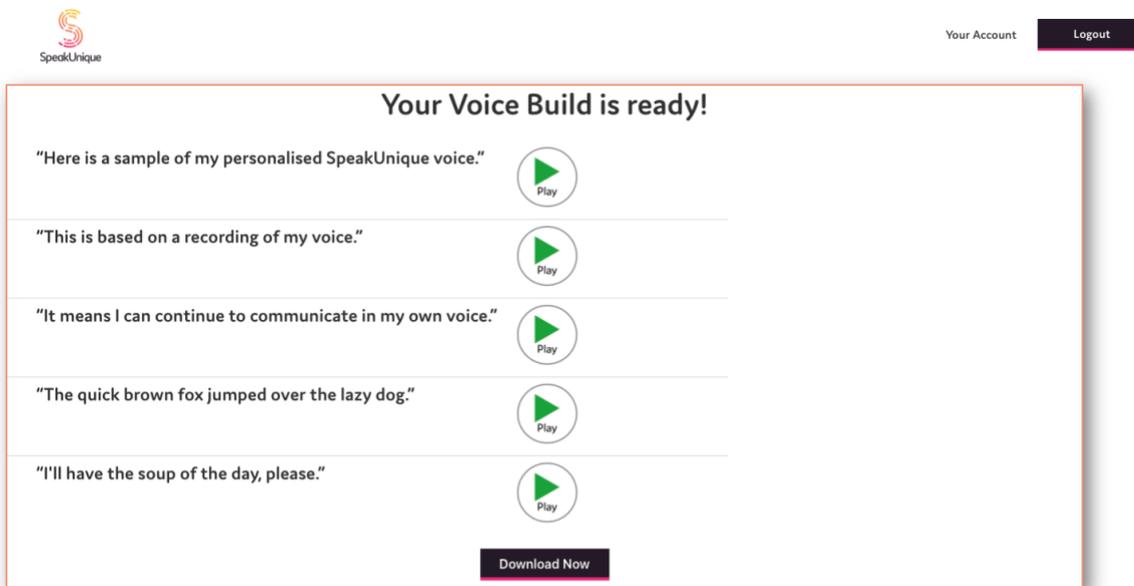

You can download your voice an unlimited number of times across multiple devices on all our supported platforms, The process for downloading your voice will depend on the device/ app you are using and specific instructions can be found by clicking on the different tabs displayed on this page.

